

Product letter to user

Thank you for using our company's products, please read the product information and usage specifications in detail, please make sure this letter reaches the end user:

Manufacturer:	Qingdao Miuton Medical Co. Ltd.
Device:	KN95 Protective Mask
model:	Ear Worn KN95 Protective Mask
Packing specifications	50pcs / box * 20boxes / carton
Meas	560mm * 380mm * 550mm
Storage	Stored in a warehouse with room temperature, dry and well ventilated environment.
Storage Life	3 years
Standard	GB 2626-2006 Respiratory protective equipment--Non-powered air-purifying particle respirator (released on March 27, 2006, implemented on December 1, 2006).
Contact Telephone	(+86)400-6717-688
Registered Address and Production address	Two / three / four floor, No. 1 Jinye Road, High-tech Industrial Development Zone, Qingdao, Shandong, China
Product URL	http://www.miuton.com/EN/ProductDetail?id=7
intended use	Reduces wearer's exposure to particles including small particle aerosols and large droplets (only non-oil aerosols). Filters out at least 95% of airborne particles including large and small particles.


Please follow the ANNEX 1 specification

Sincerely yours,

Qingdao Miuton Medical Co. Ltd.

[Instruction for use]

1. Place fingers inside the KN95 protective mask. Bend the nose clip around fingers to form nose shape.
2. Hold the KN95 protective mask in position over the nose and mouth, and pull the head harness behind the ears.
3. Using both hands, form the nose clip around the nose and shift the head harness to ensure a positive face seal.
4. Fit check a) To test the fit of the KN95 protective mask without an exhalation valve, cup both hands over mask and inhale sharply. b) If air flow is felt in the nose area, re-adjust the noses clip. c) If are flows is felt around the edges of the respirator, re-position the KN95 protective mask to achieve a better fit.
5. Change the mask immediately if breathing becomes difficult or mask becomes damaged or distorted.
6. Change the mask if a proper face seal cannot be achieved.
7. Careful observances of these instructions is an important step in safe respirator use.

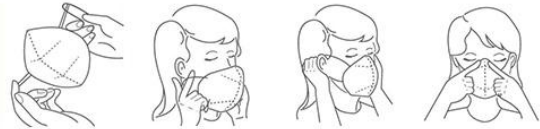

ANNEX 1

OPERATING INSTRUCTION

[Product Name] KN95 Protective mask [Product Specifications] Ear-worn

[Scope of Application] Achieve a very efficient filtration of airborne particles.

[Main Structure & Composition]

KN95 Protective mask is composed of surface layer filtering layer inner layer, nasal splint, and mask bandage. The surface layer and the inner layer are made of spunbond non-woven fabric. The filtering layer is made of melt-blown non-woven fabric. The nasal splint is made of bendable plastic. The mask bandage is made of elastic material such as non-woven fabric or rubber.

[Recommended Life Time] No more than 72 hours

[Storage] stored in a warehouse with room temperature, dry and well ventilated environment.

[Notes, Warnings & Tips]

(1) The product is not suitable for use in hypoxic environments, and is limited to daily protective use.

(2) Do not touch liquid or flame before using the product.

(3) Please stop using the product immediately if you have any discomfort during using. [Executive Standard] GB 2626-2006KN95

[Storage Life] 3 years

[Manufacturer] Qingdao Miuton Medical Co., LTD [Contact Telephone] 400-6717-688

[Date of compilation of Instruction Manual] December 10, 2019

[Registered Address and Production address] Two/three/four floor, No.1 Jinye Road, High-tech Industrial Development Zone, Qingdao, Shandong, China

